

OJOS Disección de esclerótica y conjuntiva

Nº DE PRODUCTO: 1684



DESCRIPCION:

El **Modelo Extraocular OJOS** simula las características anatómicas externas del ojo, incluyendo la conjuntiva, la esclerótica, la córnea, las inserciones del músculo recto y el nervio óptico. Este modelo permite la demostración y la práctica de múltiples técnicas que requieren **disección y manipulación de la conjuntiva y la esclerótica**.

Utilícelo con SOPORTE FLEX ORBIT para obtener los mejores resultados.

HABILIDADES:

- Conjunctiva dissection
- Scleral dissection
- Suturing

MORE THAN SIMULATORS S.L.

Calle de la Hoya, 4 2º
44570 Calanda (Teruel)
España

CONTACTO

sales@morethansimulators.com
(+34) 936 06 77 05
<https://morethansimulators.com>



- Sub-conjunctival and trans-scleral injections (some types)
- Instrument maneuvering skills

CARACTERISTICAS:

INSTRUCCIONES FOR USE WITH FLEX-ORBIT HOLDER:

This model does not require the use of the Flex Orbit Socket adapter.

- Open all the FLEX-ORBIT screws and lubricate cavity FLEX-ORBIT surfaces.
- Insert the OJOS model with the OPTIC NERVE on the nasal side. Lubricate the model.
- Secure the BASE of the eye model to the FLEX-ORBIT with the posterior screws.
- Confirm RECTUS MUSCLES are positioned properly so the screws pass in between (see figure).
- Keep anterior screws retracted. This allows for the OJOS model to move freely inside the FLEX-ORBIT, without restriction. This allows simulation of traction techniques with sutures or muscle hooks.
- Proceed to perform surgical procedure. Once surgical task is complete, open FLEX-ORBIT screws and remove the OJOS model. Lift the suction release tab to remove FLEX-ORBIT from surface. Do not pull the orbit.

INCLUYE:

- 2 cirugía extra-ocular con simulador de ojos conjuntivos sueltos
- 1 caja de almacenamiento

MORE THAN SIMULATORS S.L.

Calle de la Hoya, 4 2º
44570 Calanda (Teruel)
España

CONTACTO

sales@morethansimulators.com
(+34) 936 06 77 05
<https://morethansimulators.com>