


# LINK Plataforma Híbrida EK

Nº DE PRODUCTO: 1760

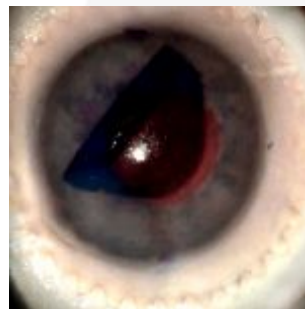

## DESCRIPCION:

**LINK** es una **plataforma de entrenamiento EK** híbrida (organosintética) que crea un modelo de cámara anterior con una **córnea donada de espesor real completa** (no incluida) y un complejo sintético de iris/esclerótica que proporciona realismo tanto funcional como estético.

LINK elimina las inconsistencias del uso de globos de investigación y brinda la oportunidad de reutilizar tejidos de calidad.

## HABILIDADES:

- Lamellar Keratoplasty escenarios (LK)
- DMEK
- PDEK
- Instrument manouvering skills

### MORE THAN SIMULATORS S.L.

Calle de la Hoya, 4 2º  
44570 Calanda (Teruel)  
España

### CONTACTO

sales@morethansimulators.com  
(+34) 936 06 77 05  
<https://morethansimulators.com>



## CARACTERÍSTICAS:

### INSTRUCCIONES FOR USE:

The system requires using a tool to consistently assemble the anterior segment with the donor cornea by crimping them with a snap "LinK" ring. To assemble:

- Lubricate the tool base posts with water or water based gel.
- Wet and slide a new LinK ring on the tool base.
- Insert and center the donor cornea inside the ring.
- Slide the anterior segment down the tool base so that it sits flush on top of the LinK ring, with the iris facing down towards the cornea.
- Insert the tool press on the tool base so that it is flush with the anterior segment, then press evenly and firmly on the press to assemble the LinK AC.
- Remove the assembled AC from the tool base and insert it into the LinK holder by gently opening the flexible holder socket.
- Fix the holder to a smooth surface with the suction cup.
- Lift the suction release tab to remove HOLDER from surface. DO NOT PULL ON THE HOLDER.

## INCLUYE:

- 1 Simulador del segmento anterior superficial
- 1 caja de almacenamiento
- \* córnea donante no incluida

### MORE THAN SIMULATORS S.L.

Calle de la Hoya, 4 2º  
44570 Calanda (Teruel)  
España

### CONTACTO

sales@morethansimulators.com  
(+34) 936 06 77 05  
<https://morethansimulators.com>